<b>Calibrador Ferritina</b>	
<b>MonlabTest®</b>	
Turbidimetría	

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### Quantitative determination of Ferritin

Only for professional *in vitro* diagnostic use  
 Store at 2-8°C

#### PRODUCT CHARACTERISTICS

The calibrator is a lyophilised serum from human origin containing a ferritin concentration suitable to calibrate ferritin determination by turbidimetry method.

#### REAGENTS

<b>Ferritin-Calibrator</b>	Calibrator. Human serum. Ferritin concentration is stated on the vial label.
----------------------------	--

#### PRECAUTIONS

Components from human origin have been tested and found negative for the presence of HBsAg, HCV and antibody to HIV (1/2). However handle cautiously as potentially infectious.

#### CALIBRATION

The concentration value of the Calibrator has been standardized against the 2<sup>nd</sup> International Ferritin Standard (85/578) from WHO (1992).

#### PREPARATION

Reconstitute (→) with 3.0 mL of distilled water. Mix thoroughly, avoiding foam forming. Bring to room temperature for about 10 minutes before use.

#### STORAGE AND STABILITY

The calibrator is stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use. Do not use reagents over the expiration date.

After reconstitution, stable for 1 month at 2-8°C or 3 months at -20°C.

#### PROCEDURE

To be used in turbidimetric assays.

#### PRESENTATION

MO-165055

1 x 3 mL