Control Microalbúmina	
MonlabTest®	
Turbidimetría	

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Quantitative determination of Microalbumin

Only for professional *in vitro* diagnostic use.
 Store at 2-8°C.

PRODUCT CHARACTERISTICS

The control is a solution containing human albumin used to evaluate the precision and accuracy of microalbumin determinations in human urine by turbidimetry method.

REAGENTS

μ ALB- Control	Human albumin solution. With a low concentration level of albumin. Preservative. The albumin concentration is stated on the vial label.
--------------------	---

PRECAUTIONS

H317 - May cause an allergic skin reaction. Contains 2-Methylisothiazol-3(2H)-one (Proclin 950). Follow the precautionary statements given in MSDS and label of the product. Components from human origin have been tested and found negative for the presence of HBsAg, HCV and antibody to HIV (1/2). However, handle cautiously as potentially infectious.

CALIBRATION

The concentration value of the Control has been standardized against the International Reference Material ERM-DA 470K/IFCC (BCR).

PREPARATION

Ready for use.

STORAGE AND STABILITY

The control is stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use. Do not use reagents over the expiration date.

PROCEDURE

To be used in turbidimetric assays.

PRESENTATION

MO-165058	1 x 2 mL
-----------	----------